


DRY EYE DISEASE AND MEIBOMIAN GLAND DYSFUNCTION

The **Tear Film** is a very thin film across the eye surface that is made up of **3 layers: oil, water, and mucus.**

The **Meibomian Glands** produce the oily layer of the tear film. In Dry Eye Disease, the oils from these glands do not flow freely, which may cause the gland openings to become plugged. Reduced oil on the surface of the eye allows tears to evaporate quickly, resulting in an uncomfortable, dry, and gritty sensation.

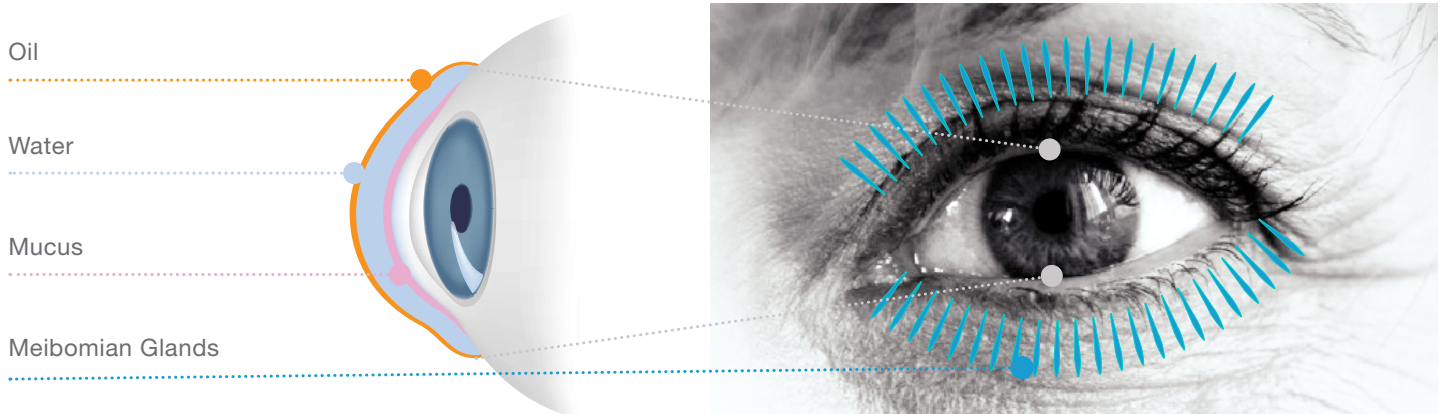


Bruder
EyeCompress
MOIST HEAT



Your Eye Care Professional has recommended you use **Bruder® Moist Heat Eye Compress** to help address your dry eye signs and symptoms.

By warming your eyelids with **Bruder® Moist Heat Eye Compress** you will help the oils in your meibomian glands to flow more freely.

Patented MediBeads® Technology

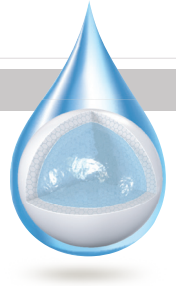

THE BRUDER® DIFFERENCE

The **Bruder® Moist Heat Eye Compress** is the **ONLY** warm compress containing **patented MediBeads® technology** where water molecules are continuously absorbed from the air and stored inside the beads. When microwaved, the absorbed water is released as clean moist heat.

The warming mask is **Hand Wash Only** in cold water with a mild detergent. **Air dry for at least 24 hours before use.**



Microwave Activated



Anti-Bacterial



Naturally Hydrating



Washable & Reusable

DIRECTIONS

- Delivers moist heat in just seconds from the microwave
- Simply place the compress in a clean microwave for **20 seconds**, then apply over closed eyes for **8 to 10 minutes** or **as recommended by your eye care professional.**



Microwave **20 Seconds**



Touch Test **Heat**



Apply To Closed Eyes **8-10 Minutes**



Air Dry At Least **24 Hours** Before Use



For additional information or questions please contact **your Eye Care Professional.**